



LOOP SYSTEM INFORMATION

DO YOU HAVE A T-COIL EQUIPPED DEVICE?

YES

- • • Activate the t-coil program



NO

- • • *Limited headsets are available. Please request one from a staff member prior to the presentation.*

1. Place the headset over both ears and adjust the side arms until you have a comfortable fit.
2. Hold the Loop Listener unit in a vertical position at head height to mimic the position of a hearing device.



3. Switch the unit on by turning the Volume Control Knob slowly clockwise until a comfortable volume is reached. Turn the knob counter clockwise to lower the volume if necessary.
4. If the LED is lit up and you are still unable to hear any sound, ask a colleague or user to speak into the microphone position on the hearing loop system, or test the input source if connected to a sound system. Recheck the Loop Listener's reception.
5. When you have finished using the Loop Listener, switch the unit off by turning the Volume Control Knob counter-clockwise until the LED is no longer lit up.

Important: Always hold the Loop Listener vertically when listening to a loop system to ensure the best possible performance.

I
DON'T
KNOW

- • • Contact your hearing professional.

If your device is equipped with a t-coil, request that it is programmed with a Manual T program and/or M/T program set for loop listening. An auto-coil will not work in loop settings so request a Manual setting that you are able to control.

If your device is not equipped with a t-coil, ask if it is possible to have one installed or added with a remote/streamer.